

User Instructions for Warm.A.Pet Flexible Heat Pad

Read through to the end before using this heat pad

- 1/. Set voltage/heat setting on the power supply to 12 volts
(Selection switch may be on the back of the power supply.If
on the underside use the key provided to change voltage up/down)
- 2/. Place heat pad in pets' bedding with some bedding material both
underneath and on top of the heat pad.
- 3/. Plug heat pad into power outlet making sure that the
two in-line connectors are fully pushed home.
- 4/. The highest setting (12 volt) may heat up quite quickly so check
the degree of warmth attained on this setting then click down
through the voltage settings to finally choose a suitable setting
(and therefore the level of warmth) that is ideal for your pet or
animal in care.

**Do Not Leave on 12 volt Setting For Any Longer Than Half an
Hour As Overheating Will *Trigger The Over-Heat Cut Out
Switch and/or create Heat Stress in the Animal in Care**

***If this occurs switch off, remove the pad from the bedding and
place pad in a cool place and allow to re-set before using again.**

**** Wildlife Carers: Always use a thermometer to monitor warmth.**

- 5/. Check the level of warmth of the pad after your pet has been in bed
for a while as the temperature will rise when your pets' body
mass and weight adds to the insulating effect of the bedding
material.

Click down or up the settings if warmth adjustments are required.

**** The bedding arrangement, body mass of your pet and ambient
temperature will affect the level of warmth attained from the heat
pad. Regularly check the pad to confirm that the chosen setting is
the most comfortable for your pet or animal in care.**

**** Power consumption when using this heat pad is minimal even
if left on 24/7.**

**** Take care of this heat pad as you would any other electrical
device and do not modify the product in any way.**

**** Manufacturing Warranty 2 Years. Contact Homark Pty Ltd Ph: 03 9544 1471**

**(Damage by chewing, moisture, accidental or wilful damage etc. not covered by any
warranty).**

View all Warm.A.Pet products and accessories at www.warmapet.com.au