

Mini-Deluxe / Multi-Voltage Heat Pad (12v, 9v, 7.5v, 6v, 5v, 4.5v, & 3volts) Switchable with key provided

Switch heat pad on to highest setting (12v) and warm up for 5 minutes only then switch down to lower setting to attain the most suitable warmth for the wildlife or pet requirement.

Caution: 12v setting will attain a very high heat quite quickly. Do not leave on this setting. Some experimentation with the settings may be required to finally choose the best setting for the animal.

Ambient temperature and weight/body mass of the animal can effect the temperature attained. In the case of care of wildlife always use a thermometer in conjunction with the heat pad to monitor the temperature of the pad to ensure the animal does not over-heat.